## DIRECTSIGHT


# Measure your pupillary distance (PD)


### **BEFORE YOU START**

1. Print out this page, **without** scaling at 100%

2. Double check the scale, by holding a credit card against the diagram below

3. Fold the ruler along the dotted line

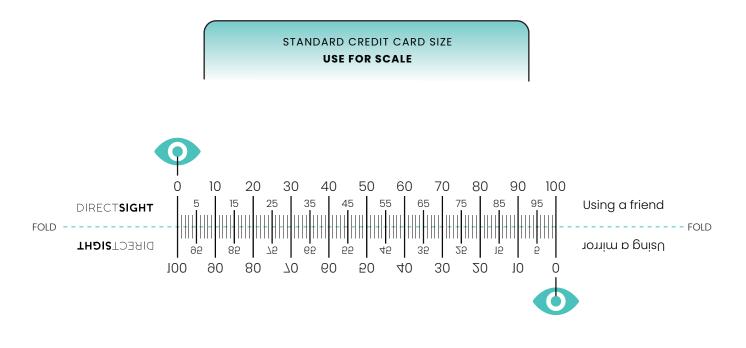

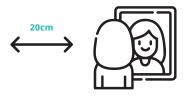


#### **STEP ONE**

Stand approximately 20cm away from a mirror. Use reversed ruler if using by yourself.

#### **STEP TWO**

Line up 0 on the ruler with the centre of your right pupil. Closing your left will ensure that the ruler is lined up perfectly with your right pupil.

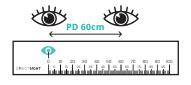

#### **STEP THREE**

Look straight ahead and measure the distance between both right and left pupils. You can repeat these steps to ensure accuracy.